ISTRUZIONI

Descrizione:

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INSTRUCTIONS

Description:

BLUEBAND is a light cure band adhesive. It is a single paste resin-based, fluoride releasing. The single paste adhesive requires no mixing; resulting in reducing adhesive waste, consistent adhesive performance and no working time constraints. BLUEBAND's distinctive blue color allows you to easily see the adhesive during banding, making clean up easy. When used with a high performance Curing Light, cure time is 20 seconds.

Indications for use:

BLUEBAND is a dental adhesive designed for banding orthodontic appliances to teeth.

Contraindications:

BLUEBAND is contraindicated in patients with a history of severe allergic reaction to methacrylate resins.

Warning:

Do not use in the presence of eugenol based materials as complete setting may not occur.

General Precautions:

1. This device must be sold by or on the order of a dentist.
2. Review instructions and MSDS prior to beginning treatment. Use only as directed.

Adverse Reaction:

Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.

Instructions for Use:

1. Thoroughly clean teeth to be cemented.
2. Rinse teeth thoroughly with large amounts of water and dry with oil and moisture-free compressed air. Provide suitable moisture control to isolate area and to prevent moisture contamination during setting.
3. For proper fit, trial fit each band to be cemented on the patient. After fit has been verified, remove band and clean.
4. Hold the band between the thumb and forefinger. Apply a bead of BLUEBAND to the inside of the band. Cover the entire inside of the band by spreading the paste with a small spatula or other suitable instrument.
5. Set band on tooth, removing excess BLUEBAND with a scaler.
6. Using a visible light-curing unit, with a minimum output of 600 mW/cm², cure the material for at least 20 seconds. The light tip should be placed in close proximity to the occlusal surface of the tooth being banded.
7. Replace cap on the BLUEBAND syringe.

Storage:

1. recommended storage conditions are between 2°C/36°F to 27°C/80°F, bringing to room temperature (22°C/72°F) before using. Do not freeze. See outer package for expire date.
2. Do not expose to elevated temperature or intense light.
3. Keep container closed when not in use.